



**ARTERIAL TOURNIQUET TRAINING
DEVICE
BLOOD OUT
USER MANUAL**

1. Place the strap around the thigh as tightly as possible.



2. Secure the strap using the Velcro fastener



3. Attach the electronic device using the Velcro



4. To prevent the strap from sliding down, secure it with the belt strap to your belt or a belt loop



5. Device buttons and LED indicators

Tourniquet pre-tension indicator. When the pre-tension is applied with sufficient force, the light turns off.
Pre-tension pressure: 50 mm/Hg.

Battery level indicator

Green: 100–50%

Orange: 50–30%

Red: 30%

Operating time with a fully charged battery: approx. 14 h



When the bleeding stops, the light turns off.
Bleeding cessation pressure: 400 mm/Hg.

ON/OFF button

Battery charging indicator

Orange: Battery is charging

Green: Battery is fully charged

Warnings

- Release the tourniquet immediately if you feel excessive pressure on the limb.
- Bloodout OÜ is not liable for any potential health damage or injuries.
- The device is splash-proof (IP44); do not use it outdoors in heavy rain.
- The warranty is valid for 6 months from the date of handover of the device. The warranty does not apply in cases of improper use of the device.
- If you have any questions, please feel free to contact:

Bloodout OÜ

info@bloodout.eu

www.bloodout.eu

